



# Bedsoft Update Tool

## Quick Start Guide

Smokerlyzer® Micro<sup>+</sup>, piCO<sup>+™</sup>

ToxCO<sup>™</sup>, Gastro<sup>+™</sup>

1. Download the Bedsoft Update Tool Program:

- a. [Click Here](#)
- b. You will be presented with two options, “run” or “save”  
Click “run” to download and start the installation or “save” to download and start the installation at a later date.
- c. I have chosen “run”



- d. You will then be presented with a security warning, click “run” to confirm you wish to continue with the installation.



- e. You have now begun the installation wizard, click “**next**” to continue with the installation.



- f. Type your “user name” and the “name of your organization” and select “**next**”.



- g. Now click **“install”** to start installing the software.




- h. You have now successfully installed the software, click **“finish”** to exit the installation wizard.




- i. To run the software you will have to double click an icon on your desktop called “**Bedfont Software Update**”, which looks like the one below.




- j. You will be presented with the window below. Click the **positive symbol “+”** highlighted in **blue** for the software that you would like to update. i.e. (COData+, ToxCOData, GastroCHART).

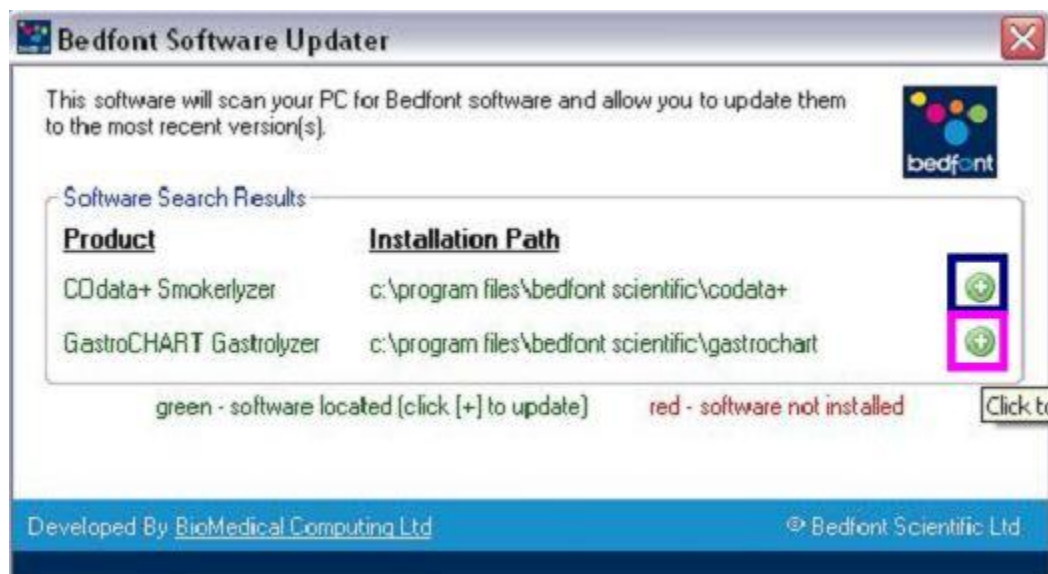

- k. In example above we selected COdata+. Please ensure that the COdata+ software is not currently running/active on your system and select “ok” to continue with the update.

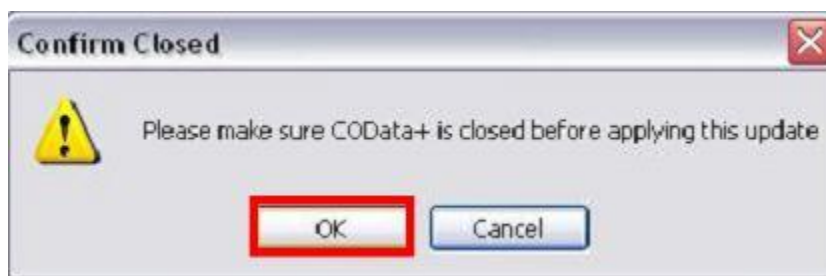

- I. Once the update is complete you will be notified via the following window below. Click “ok” to remove this message and close the update tool using the close button “red X” in the top right of the window.

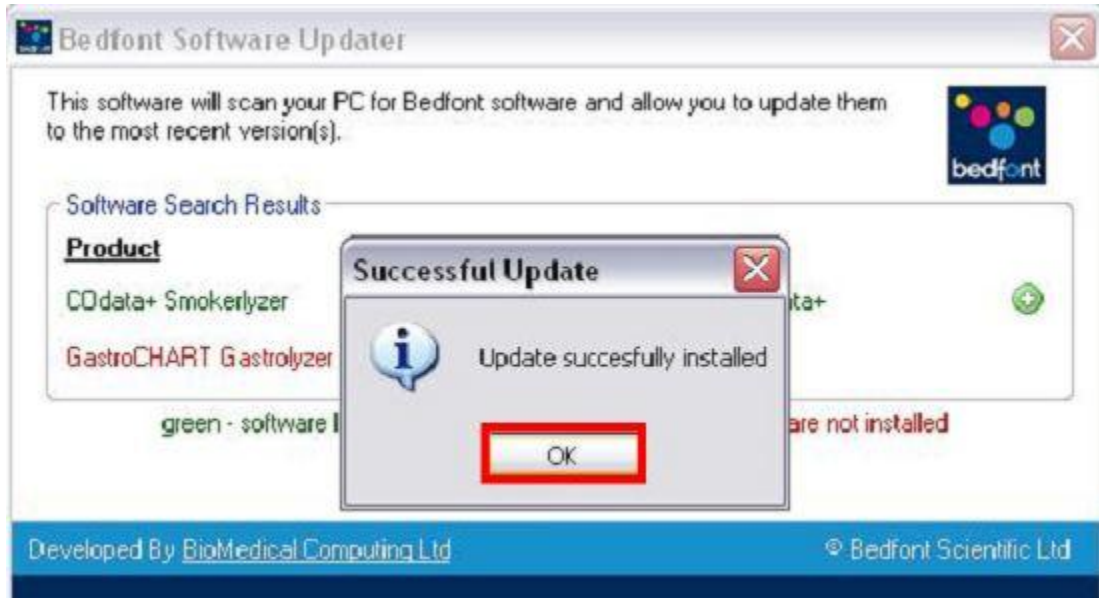

- m. Your software is now up to date and you can continue to use it as before.

For questions please contact us:

**coVita**

**Customer Service**

tel (800) 707-5751

fax (800) 721-2377

email: [service@covita.net](mailto:service@covita.net)

[www.covita.net](http://www.covita.net)