



Calibration Unlock Tool

Quick Start Guide

Smokerlyzer[®] Micro⁺, piCO⁺TM

ToxCOTM, Gastro⁺TM Gastrolyzer[®]

TIPS: You will need to have admin rights on your computer to install this program. Any questions please contact you IT department.

1. **Download the Calibration Unlock Tool Program:**

- a. [Click Here](#)
- b. You will be presented with two options, “run” or “save”
Click “run” to download and start the installation or “save” to download and start the installation at a later date.
- c. I have chosen “run”



- d. You will then be presented with a security warning, click “run” to confirm you wish to continue with the installation.




- e. You have now begun the installation wizard, click “**next**” to continue with the installation.



- f. Now click “**install**” to start installing the software.



- g. You have now successfully installed the software, click “**finish**” to exit the installation wizard.

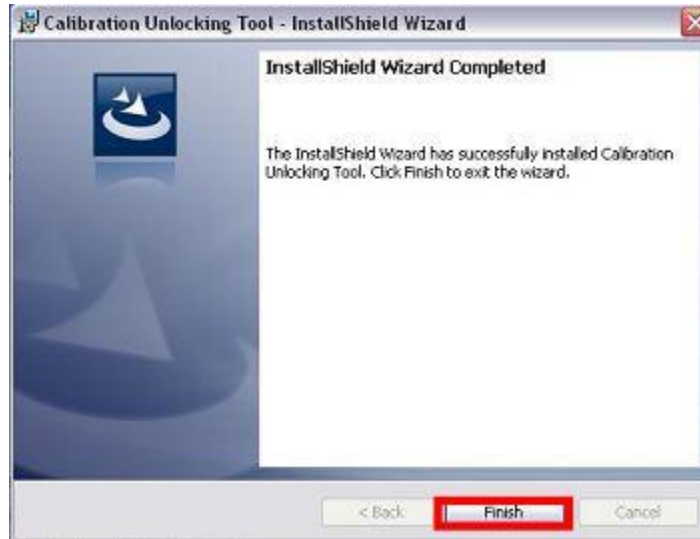

- h. You will need to **plug** your device into the computer using the mini USB lead and turn it on to install the driver. Your computer should automatically detect and install your device. Please wait for the **drivers** to install before continuing.



- i. Now double click the “**Calibration Unlocking Tool icon**” on your desktop.



- j. The Calibration Unlocking Tool program is now launched. As you can see you have three options: Smokerlyzer, Gastrolyzer and ToxCO. **Click the type of device that you have.**



- k. Now simply click “**Unlock Device**” to unlock the calibration mode on your device.
- l. You have successfully unlocked the calibration mode on your device, click “**OK**” to close the information box (pictured below). You may then **remove/unplug** the USB cable from your device and computer.



- m. **Restart** your device before calibrating.
- n. **Calibrate** your monitor as you would normally.
- o. Please note that it is **recommended** that you leave this software installed on your computer in case you need to unlock your device again.



For questions please contact us:

coVita

Customer Service

tel (800) 707-5751

fax (800) 721-2377

email: service@covita.net

www.covita.net