

# GastroCH<sub>4</sub>ECK<sup>®</sup> Gastrolyzer<sup>®</sup>

## INSTALLATION & CAUTIONS

### Installation

#### On receipt of a new GastroCH<sub>4</sub>ECK<sup>®</sup>:

1. Remove the unit from the packaging supplied to you
2. Place the unit in the location it is most likely to be used and ensure a clearance of 30cm (1ft) above and 15cm (6in) at the sides and rear of the GastroCH<sub>4</sub>ECK<sup>®</sup>
3. **DO NOT** switch on for the first 24 hours
4. **After 24 hours**, switch the device on and allow  $\leq 2$  minutes for the GastroCH<sub>4</sub>ECK<sup>®</sup> to start up
5. Please allow the GastroCH<sub>4</sub>ECK<sup>®</sup> to be switched on for at least 5 minutes prior to calibration.

### Cautions

- Do not use this device outside the operational temperatures of between 15-35°C. If the device has been exposed to temperatures outside of the 15-35°C range, then please leave in an area within the range for a minimum of 6 hours.
- Do not use alcohol based hand cleaners before using this device. If the device has been exposed to alcohol based hand cleaners, then please leave for 6 hours for the device sensors to settle before using or calibrating the device.
- The GastroCH<sub>4</sub>ECK<sup>®</sup> should be switched off once every 24 hours.



#### USA Sales Service and Support



[www.covita.net](http://www.covita.net) | 800.707.5751 | [info@covita.net](mailto:info@covita.net)



Visit [www.bedfont.com/resources](http://www.bedfont.com/resources) to view this document in other languages.



Bedfont Scientific Ltd.  
Station Road, Harrietsham, Maidstone  
Kent, ME17 1JA, England.  
Tel: +44 (0)1622 851122, Fax: +44 (0)1622 854860  
Email: [ask@bedfont.com](mailto:ask@bedfont.com) Web: [www.bedfont.com](http://www.bedfont.com)

© Bedfont\* Scientific Limited 221

Issue 3- October 2021, Part No: LAB742\_AX  
Bedfont\* Scientific Limited reserve the right to change or update this literature without prior notice.  
Registered in: England and Wales. Registered No: 1289798



Stephen Rowe  
Cristimar E4-1  
Ave Juan Carlos I  
Los Cristianos, Arona, 38650  
Santa Cruz de Tenerife, Spain

