

Smokerlyzer®

Helping people to stop smoking one breath at a time.™

Use & Storage



www.covita.net | 800.707.5751

Copyright 2023 coVita LLC All Rights Reserved v042123



Storage & Use

- Only use coVita alcohol-free hand sanitizers and wipes or approved products on or around the monitors to avoid interference with the monitor's electrochemical sensors. If you must use alcohol-based hand sanitizers on your hands, allow your hands to dry completely before handling the monitors.
- Be aware that your monitor is sensitive to vapors from VOCs (Volatile Organic Compounds) such as alcohol based hand sanitizers and wipes, fresh paint, fumes from solvents. Be aware of the environment that you are using your monitor in. Prolonged exposure to vapors or fumes of any kind may impact your monitor. Remove it from these adverse situations.
- As with any electronic devices **never** store or leave the monitors in environments (i.e. cars etc.) with extreme temperatures (hot or cold) for extended period of time.
- If you do not plan on using your monitor within 30 days, please remove the batteries. Also it is recommended to only use alkaline batteries. Please note if rechargeable batteries are used it is advised to remove the batteries when not in use because they generally have a higher probability of leaking.
- **Never** block the air/vent hole on the Smokerlyzer® with labels.

Accessories



7. SteriBreath™
Mouthpiece
Single Use



3. D-piece™
One way valve
replaced monthly



12. Calibration Kit
Calibration required
every 6 months

Monitor Layout

1. Display
2. Power Button
3. D-piece™ One way valve
4. D-piece™ Port
5. Exhaust port—**Do not cover with labels**
6. USB connector for use with Software
7. SteriBreath™ Mouthpiece
8. Battery Compartment—3 AA Batteries
9. Battery Compartment Opening Clip
10. Reset Button
11. Programming Switch
12. **20** ppm **CO** Calibration Kit

