

# Infection Control and Maintenance Guidelines



## For the Bedfont® NObreath® device and NObreath® Dock

Before using any of the above equipment, the Operating Manual should be read and fully understood.

### Infection Control

Washing hands before and after testing is highly recommended for both operator and user as part of a sensible infection control regime.

The NObreath® FeNO device is integrated with antimicrobial technology for improved infection control.

**WARNING:** The mouthpieces are single patient use only and can be used for a maximum of 10 tests per breath testing session. Further re-use could cause incorrect readings and could increase the risk of cross infection. The mouthpiece should be disposed of after use, in accordance with local waste disposal guidance.

The NObreath® mouthpiece contains an integrated infection control filter that removes and traps >99% of airborne bacteria and >98% of viruses<sup>1</sup>.

Take advice from your infection control expert before use on patients with known communicable diseases/conditions.

It is preferable that the user attaches their own mouthpiece to the device before the breath test, and detaches and disposes of it once the test is complete.

### Routine maintenance

See the operating manual for full maintenance instructions.

- The NObreath® should be serviced annually
- It is best practice to not let the battery run flat
- You should perform a quality check every 6 months



Bedfont® Scientific Ltd.  
Station Road, Harrietsham, Maidstone,  
Kent, ME17 1JA England  
Tel: +44 (0)1622 851122 Fax: +44 (0)1622 854860  
Email: ask@bedfont.com Web: www.bedfont.com

### USA Representative for Sales, Service & Support:



212 Cottage Grove Ave Suite C  
Santa Barbara, CA 93101  
Tel: 800.707.5751 Fax: 800.721.2377  
Email: service@covita.net Web: www.covita.net

Stephen Rowe  
Cristimar E4-1  
Ave Juan Carlos I  
Los Cristianos, Arona, 38650  
Santa Cruz de Tenerife, Spain

© Bedfont® Scientific Limited 2023

Issue 9 - July 2023, Part No: LAB766\_USA  
Bedfont® Scientific Limited reserves the right to change or update his literature without prior notice.  
Registered in: England and Wales. Registered No: 1289798

Rx only



MID 502905

# Infection Control and Maintenance Guidelines



## Cleaning

Bedfont® recommends wiping the instrument's external surfaces with a product specifically developed for this purpose. An EPA registered alcohol free disinfectant should be used, such as Sani-Cloth AF3 Germicidal Disposable Wipe. Cleaning should be performed following the instructions for use specified by the manufacturer of the EPA registered disinfectant. The device or consumables cannot be sterilized. It is recommended that wipes are used once and for one surface only. The NObreath® device should be cleaned for initial use and after each patient use.

**CAUTION: Do not use any substances containing alcohol on or near the NObreath®.**

**NEVER** use alcohol or cleaning products containing alcohol or other organic solvents as these vapours will damage the sensor within the instrument.

**WARNING:** Under no circumstances should the instrument be immersed in or splashed with liquid.

This document is for guidance only.

## REFERENCES

1. Public Health England. An Evaluation of Filtration Efficiencies Against Bacterial and Viral Aerosol Challenges. London: Public Health England; 2017.



**Bedfont® Scientific Ltd.**  
Station Road, Harrietsham, Maidstone,  
Kent, ME17 1JA England  
Tel: +44 (0)1622 851122 Fax: +44 (0)1622 854860  
Email: ask@bedfont.com Web: www.bedfont.com

## USA Representative for Sales, Service & Support:



212 Cottage Grove Ave Suite C  
Santa Barbara, CA 93101  
Tel: 800.707.5751 Fax: 800.721.2377  
Email: service@covita.net Web: www.covita.net



**Stephen Rowe**  
Cristimar E4-1  
Ave Juan Carlos I  
Los Cristianos, Arona, 38650  
Santa Cruz de Tenerife, Spain

© Bedfont® Scientific Limited 2023

Issue 9 - July 2023, Part No: LAB766\_USA

Bedfont® Scientific Limited reserves the right to change or update his literature without prior notice.  
Registered in: England and Wales. Registered No: 1289798



MD 502905