




1. To send all current readings from the Smokerlyzer® app
 - a. **Select results**

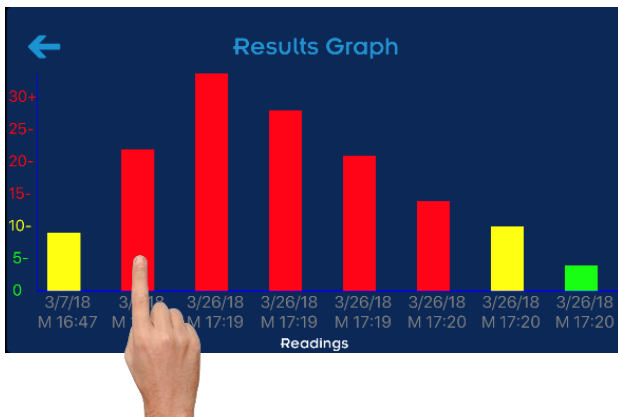


2. **Select** any one of the *bars in the graph* which will then take you to a new screen which will allow you to share either the *single result* previously clicked on **or** *all of the logged results*.

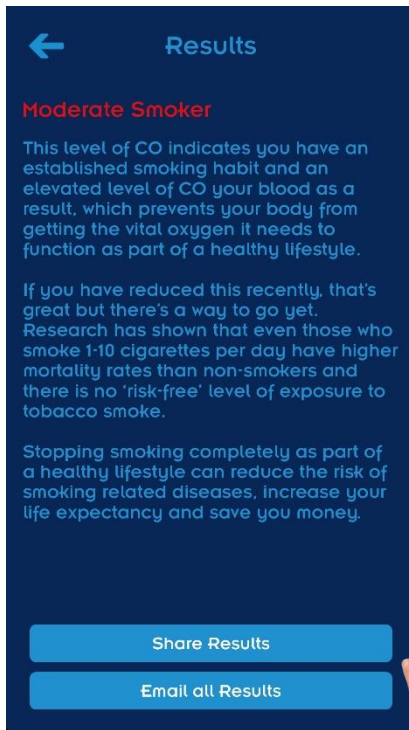

3. **Select Either:**
- a. *Share Results*
 - i. Share the individual result that you was previously clicked on
- OR**
- b. *Email all results*
 - i. Share all logged results

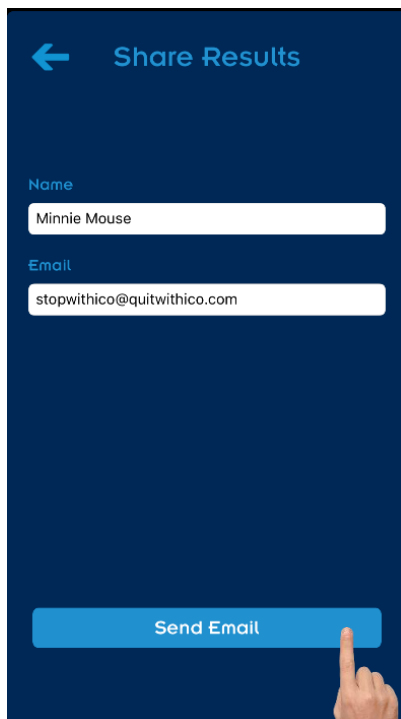

4. Verify email address and **select send email**

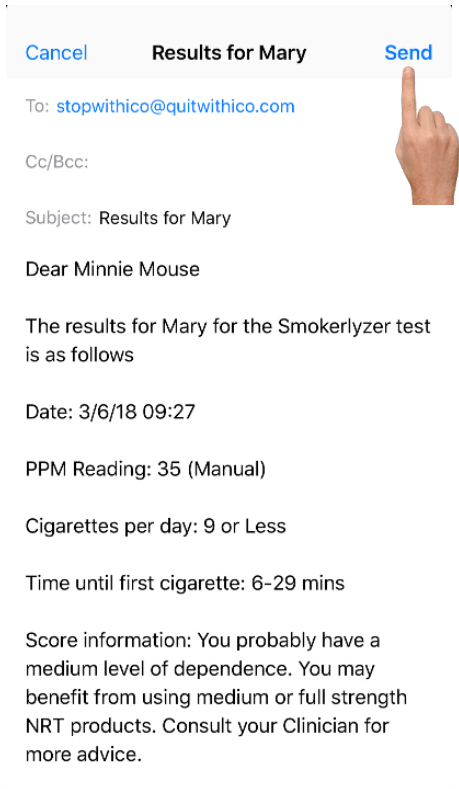

5. Select *send email*

- a. All readings will be emailed with the dates, times ppm readings, and device serial numbers
- b. Answers to questions will also be emailed if the questions are enabled
- c. Done!