

Testing Process

Conducting a breath test is simple, non-invasive, and instant.

1

DOWNLOAD THE ICOQUIT® APP OR ONE OF OUR PARTNER APPS



2

LAUNCH THE SMOKERLYZER® APP & FOLLOW INSTRUCTIONS




3

CONNECT ICOQUIT® TO YOUR SMARTPHONE OR TABLET



4

TAKE A DEEP BREATH

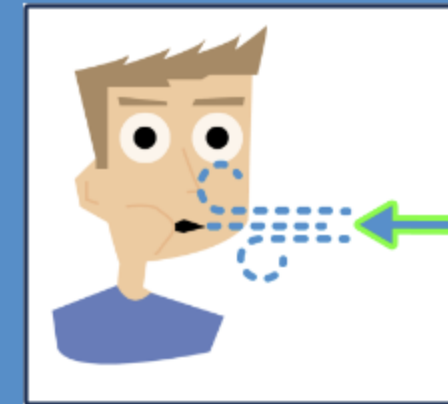


5

HOLD BREATH FOR 15 SECONDS



6

EXHALE INTO ICOQUIT®

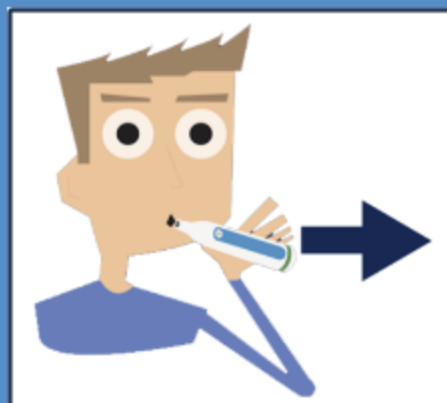

7

INSTANT READING




8

OPTION TO SHARE RESULTS



9

AUTOMATICALLY SAVES AND CHART RESULTS

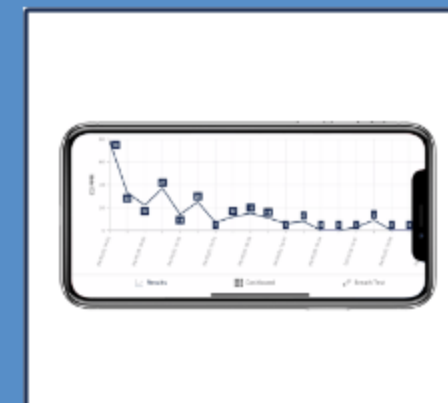

10

BEGIN YOUR JOURNEY TO A SMOKE FREE LIFE

